

Xpert[®] HIV-1 VL Cartridge Preparation

Xpert[®] HIV-1 Viral Load

Refer to the package insert for detailed instructions, precautions, and warnings.

For a copy of the SDS, visit www.cepheid.com or www.cepheidinternational.com
Cepheid Technical Support

US office
(888) 838-3222, Option 2
techsupport@cepheid.com

European office
+33 563 825 319
support@cepheideurope.com

Prior to beginning this test:

1. Centrifuge the sample to separate the plasma and red blood cells per the manufacturer's instruction.
2. Equilibrate the plasma to room temperature (20-35 °C).
3. Vortex plasma for 15 seconds. If the specimen is cloudy, clarify with a quick spin.


301-4647 Rev. B March 2024 2797




- 1 Take one Xpert cartridge for each sample.



- 2 Open the cartridge lid.

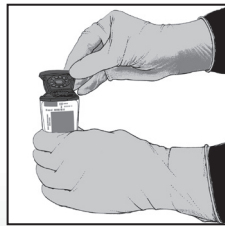

- 3 Aspirate the plasma to just above the 1mL mark on the pipette.

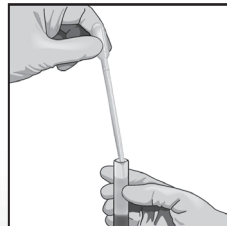

- 4 Empty the contents into the sample chamber.



- 5 Close the cartridge lid.



- 6 Start the test within the timeframe specified in the package insert.